



Safescraper® TWIST Cortical Bone Collector



A New Twist On Manually Harvesting Bone

- **Provides 160° Cutting Area – Effective On Plane, Concave And Convex Bone Surfaces**
- **Available In Curved And Straight Designs – Versatile, Facilitating Access To Hard-To-Reach Posterior Regions**
- **Large Capacity Chamber – Minimizes Bone Shaving Dispersion And Retains Bone In A Sterile Chamber To Limit Patient Contamination**
- **Lateral Opening System – Provides Device Stability And May Be Used As A Container For Composite Grafts**
- **Minimally Invasive – The Manual Harvesting Technique Reduces Operating Time And Post-Operative Patient Discomfort**



Safescraper® TWIST: How It Works

1. The bone is collected by pushing the end of the device toward the bone surface with the chamber facing downward and simultaneously pulling the device itself backward.
2. To retrieve collected bone, keep the transparent chamber facing downward, press the lock button sideways and slide the chamber clockwise.

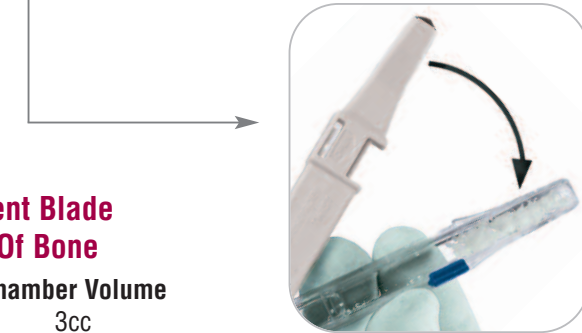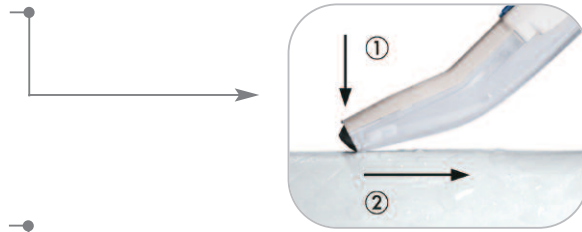

Each Safescraper TWIST Has A Per-Patient Blade Life That Effectively Harvests Up To 5cc Of Bone

Ordering Codes	Chamber Volume
3598 Safescraper TWIST Straight - 3 units	3cc
3987 Safescraper TWIST Curve - 3 units	2.5cc

Shelf life of Safescraper TWIST is three years.

Get A New Twist For Your Bone Graft And Defect Cases By Contacting Your BIOMET 3i Sales Representative Today.

Distributed by:



Global Headquarters
4555 Riverside Drive
Palm Beach Gardens, FL 33410
1-800-342-5454
Outside The U.S.: +1-561-776-6700
Fax: +1-561-776-1272
www.biomet3i.com

Sign Up For BIOMET 3i's Electronic Newsletter
"BIOMET 3innovations." Simply Go Online To
www.biomet3i.com/signup



ART819

REV C 01/08

Safescraper is a registered trademark of CGM S.p.A. Divisione Medica META. BIOMET is a registered trademark and BIOMET 3i and design are trademarks of BIOMET, Inc. ©2008 BIOMET 3i, Inc. All rights reserved.

SUBSIDIARIES

AUSTRALIA
Phone: +61-2-9855-4444
Fax: +61-2-9888-9900

BRAZIL
Phone: +55-11-5081-4405
Fax: +55-11-5081-7484

CANADA
Phone: +514-956-9843
Fax: +514-956-9844

FRANCE
Phone: +33-1-41054343
Fax: +33-1-41054340

IRELAND
Phone: +35-31-477-3925
Fax: +35-31-402-9590

MEXICO
Phone: +52-55-5679-1619
Fax: +52-55-5684-8098

THE NETHERLANDS
Phone: +31-(0)78-629-2800
Fax: +31-(0)78-629-2801

NEW ZEALAND
Phone: +64-508-122-221
Fax: +64-508-133-331

NORDIC REGION
Phone: +46-40-17-6090
Fax: +46-40-17-6099

PORTUGAL
Phone: +351-21-000-1645
Fax: +351-21-000-1675

SPAIN
Phone: +34-93-470-59-50
Fax: +34-93-372-11-25

U.K.
Phone: +44-1628-829314
Fax: +44-1628-820182

DISTRIBUTORS

ARGENTINA
Dentalmax, SA
Phone: +541-1482-71001
Fax: +541-1482-67373

CHILE
Cybel, SA
Phone: +56-2-2321883
Fax: +56-2-2330176

CHINA
Atek Inc.
Phone: +86-21-6329-1265
Fax: +86-21-6329-1620

COLOMBIA
3i Colombia
Phone: +571-612-9362
Fax: +571-620-6412

COSTA RICA
Implantec S.A.
Phone: +506-2-256411
Fax: +506-2-247620

EL SALVADOR
Dentimerc SA de CV
Phone: +503-263-6350
Fax: +503-263-6676

GREECE
Kostas Kornisorlis and Co.
Phone: +30-2310-501-651
Fax: +30-2310-522-417

HONG KONG
Ositek Inc., Ltd.
Phone: +852-8121-6601
Fax: +852-3747-3754

PARAGUAY
Andres H. Arca y Cia SRL
Phone: +595-21-208185
Fax: +595-21-496291

POLAND
Dental Depot
Phone: +48-71-341-3091
Fax: +48-71-343-6560

RUSSIA
Com-Dental
Phone: +7-495-797-6686
Fax: +7-499-242-9567

SINGAPORE
Asia Implant Support & Services
Phone: +65-6223-2229
Fax: +65-6220-3538

SOUTH AFRICA
Selective Surgical CC
Phone: +27-11-991-7007
Fax: +27-11-672-1391

THAILAND
3i (Thailand) Co., LTD.
Phone: +662-252-6685
Fax: +662-252-6686

UKRAINE
Com-Dental
Phone: +38-067-7007667
Fax: +38-044-5017117

URUGUAY
Pro3implant S.R.L.
Phone: +598-2-4034163
Fax: +598-2-4034163